

Mölnlycke[®] Exufiber[®]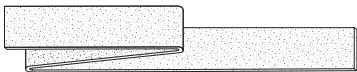

STERILE EO






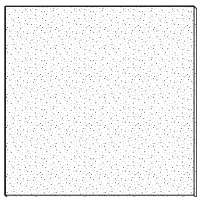
Manufacturer

Mölnlycke Health Care AB
Gamlestadvägen 3C, Box 13080
SE-402 52 Göteborg, Sweden

www.molnlycke.com

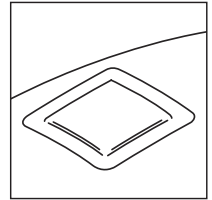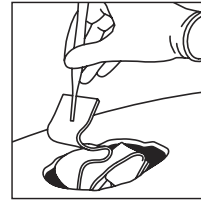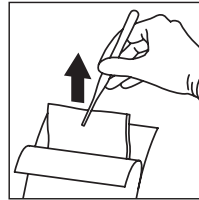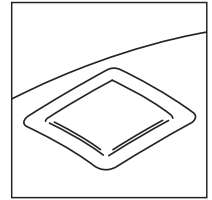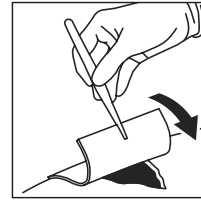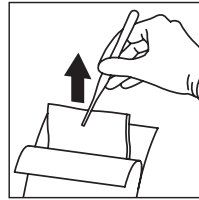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Product description

Exufiber is a sterile nonwoven dressing made from highly absorbent polyvinyl alcohol (PVA) fibres. In contact with wound exudate, Exufiber transforms into a gel that facilitates moist wound healing and ease of removal during dressing change. Exufiber absorbs and retains wound exudate. Exufiber is available both as pad and ribbon dressings.

Indications for use

Exufiber wound dressing is intended to be used on a wide range of exuding wounds:

- Leg and foot ulcers
- Pressure ulcers
- Partial thickness burns
- Surgical wounds
- Donor sites
- Malignant wounds

Precautions

- Do not use on patients with known hypersensitivity to the dressing or its materials.
- Should infection develop during the use of the dressing, medical treatment should be initiated according to local clinical protocols. Infection is not a contraindication to the use of Exufiber dressing.
- Exufiber is not intended for dry wounds or full thickness burns.
- Exufiber is for single-use only and should not be re-used. Re-use may lead to product deterioration or cross contamination may occur.
- Sterile. Do not use if inner package is damaged or opened prior to use. Do not re-sterilise.

Instructions for use

Exufiber should be used by, or under the supervision of, a qualified health care professional.

Application*Shallow wounds*

Cleanse the wound with saline solution or water according to clinical practice.

1. Dry the surrounding skin thoroughly.
2. Choose the correct size of dressing to be able to cover the entire wound. For best result, Exufiber should overlap the dry surrounding skin by at least 1-2 cm for smaller product sizes, (up to 10x10 cm) and 5 cm for larger sizes. The dressing will swell, causing contraction of the dressing edges, as it absorbs wound fluid and starts gelling.

3. Apply a dry Exufiber dressing to the wound.
4. Fixate with an appropriate secondary dressing. The choice depends on the exudate level.
5. Compression therapy may be used in conjunction with Exufiber.

Deep wounds (cavities, tunnelling or undermining)

Cleanse the wound with saline solution or water according to clinical practice.

1. Dry the surrounding skin thoroughly.
2. Loosely pack ribbon or pad into the wound to allow room for swelling of the dressing. When using the ribbon dressing, cut to appropriate length leaving a small overhang of 2-3 cm outside the wound for easy retrieval.
3. Cover with an appropriate secondary dressing. The choice depends on the exudate level.
4. Fixate (or cover) with an adhesive dressing or a bandage as appropriate.

Donor sites

Harvest tissue and ensure hemostasis.

1. Choose the correct size of dressing to be able to cover the entire wound. The Exufiber dressing should overlap the dry surrounding skin by at least 5 cm. The dressing will swell, causing contraction of the dressing edges, as it absorbs wound fluid and starts gelling.
2. Apply a dry Exufiber dressing to the wound.
3. Cover with an appropriate secondary dressing. The choice depends on the exudate level.
4. Fixate (or cover) with an adhesive dressing or a bandage as appropriate.

Dressing change and removal:

Change the Exufiber dressing when fully saturated, at signs of leakage, or as indicated by clinical practice. Exufiber can be left in place for up to 7 days (14 days in donor sites) depending on wound condition and clinical practice.

1. Remove the secondary dressing.
2. Remove the Exufiber dressing by gently cleansing/flushing if required. Any non-gelled material will moisten in contact with fluid.
3. If the dressing dries out and is difficult to remove, it should be moistened with saline solution or water according to clinical practice and allowed to soak until it lifts easily. It may take several minutes for Exufiber to transform into a gel. Remove the dressing by gently cleansing/flushing.
4. Disposal should be handled according to local environmental procedures.

Other information

If any serious incident has occurred in relation to the use of Exufiber it should be reported to Mölnlycke Health Care.

Exufiber® is a registered trademark of Mölnlycke Health Care AB.

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Exufiber is a registered trademark of Mölnlycke Health Care AB